

The **YES** Company!!



**SPILL KIT**  
Absorbant Clean-up Kit

- CONVENIENT ALL IN ONE KIT

KIT INCLUDES:  
 ABSORBENT POWDER, LATEX GLOVES, DISPOSABLE PLASTIC BAGS, DISPOSABLE SCOOPS, GERMICIDAL WIPE, ANTISEPTIC WIPES & INSTRUCTION SHEET

**Product # 7176**

**FOR PROFESSIONAL USE ONLY**  
 CONSULT LABEL AND MSDS BEFORE USING.

1	HEALTH
0	FLAMMABILITY
0	REACTIVITY
	SPECIFIC

INGREDIENTS	CAS NUMBER
ALUMINUM SILICATE	1332-58-7
CORN STARCH	9005-25-8
AMORPHOUS SILICA	112926-00-8
AMYL ACETATE	628-63-7
ETHYL ACETATE	141-78-6

Additional Information  
 The **SUPERCO SPILL KIT** is the latest in absorbant spill pickup technology. The **SUPERCO SPILL KIT** is a completely self contained kit with everything necessary for small spill pickup. The absorbant quickly absorbs 80-100 times its own weight and congeals into a easy to handle gel for easier disposal. For more information on **SUPERCO'S SPILL KIT**, contact your local representative.

**SPILL KIT**  
**ABSORBANT CLEAN-UP KIT**

The fear of handling spilled body fluids has increased markedly in recent years. That is why there should be a **SUPERCO SPILL KIT** wherever bodily spills are likely to occur. The **SUPERCO SPILL KIT** is a complete, ready to use absorbant clean-up kit that has everything necessary to easily contain small spills. The absorbant powder will absorb 80-100 times its own weight. Ideal for Schools, buses, hospitals, and nursing homes.

**DIRECTIONS FOR USE:**

1. Put on disposable latex gloves immediately.
2. Cover the spill with the absorbant powder(will absorb 80-100 times its own weight). Allow to thicken into a gel and using the two disposable scoops(one in each hand), remove the gelled waste material from the spill area and place in the first black disposable bag.
3. Close the first black disposable bag with twist tie and place the sealed bag into the second black disposable bag.
4. Using the germicidal disposable wipe, clean up the spill area and place this in the second black disposable bag.
5. Carefully remove the latex gloves and discard in the remaining disposable bag.
6. Open antiseptic towelette package and thoroughly wash your hands. Place the used towelette in the plastic bag.
7. Close the second black disposable bag with the twist tie and place in the red biohazard bag.
8. Close the red biohazard bag with the twist tie and discard in accordance with local regulations.